

## EVO VELOCITY TRIO Quick Start Guide

CE 0123  
Directive 93/42/EEC  
concerning Medical Devices



### 1 Applicator installation / Removal

#### Installation:

Insert the applicator vertically into the socket (applicator knob as shown in the photo), rotate the knob clockwise (about 300 degrees) to the red mark, then the applicator is fixed.

#### Removal:

Rotate the applicator knob counterclockwise to the horizontal state and gently unplug it.

### 2 Adding water / Draining water

#### Adding water:

Insert the water filling funnel into the "inlet" at the back of the machine, and then add about 2.5 liters of deionized water or purified water. Please don't exceed the maximum water level. If it exceeds the water level, which would flow from the "vent". Water level can be observed in real time from the observation window.



#### Draining water:

Connect the drain pipe to the drain outlet and the water flows out automatically.

#### Installation:




#### Removal:

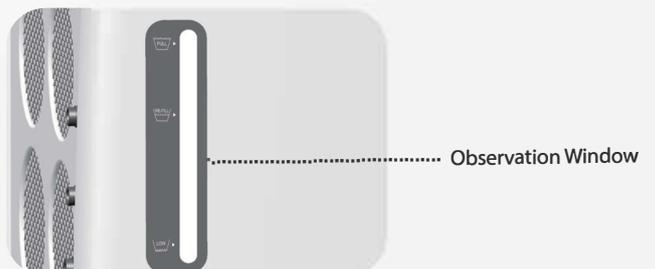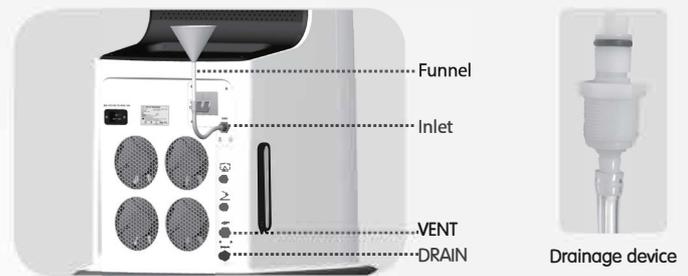

### 3 Door interlock

After the door interlock is disassembled, welded it with the electronic switch of the treatment room with a wire. The electronic switch of the treatment room is welded with a wire. Then insert the door interlock plug into the socket shown in the photo. (This is a safety measure and should be installed by a professional.) If it cannot be connected to the electronic switch, please use the factory-supplied accessories.

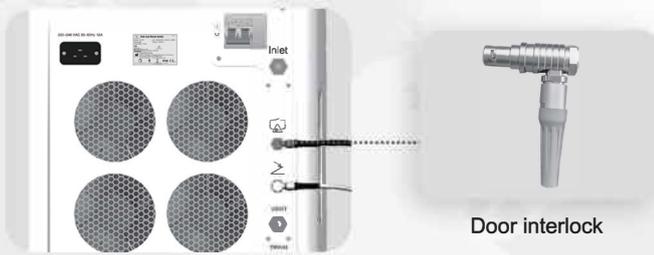


### 4 Foot switch

Plug the 3-pin aviation plug that connecting the foot switch into the plughole. (Optional)



### 5 Power Cord

Plug one end of the power cord into the host machine plughole, and connect the other end to the socket.

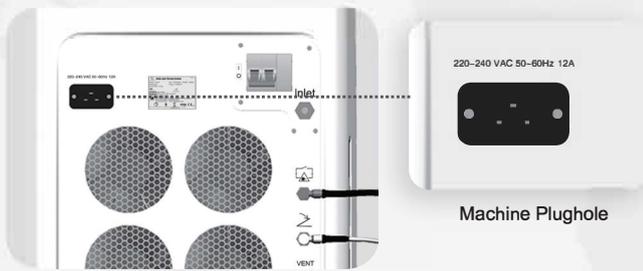

### 6 Air switch

Turn on the air switch on the rear side of the machine.



### 7 Password

Press the power button on the front left side to start the machine. After the machine start-up, the interface will pop up the password box automatically. Enter the correct password (password:123456), then system starts the self-checking program.



### 8 Professional interface

After the self-checking program is done, automatically enter the professional interface.



### 9 Working mode switch

Click the setting  button to enter the setting interface, switch to the aesthetic interface.



### 10 Applicator display

Applicator with real-time OLED screen can display treatment parameters or process.



#### Note:

1. The applicator contains precision optical components, please take it gently.
2. Make sure to install the door interlock, to avoid the injury.
3. It is prohibited to observe the laser beam output from the applicator directly.

4. Please replace the water in the tank every month (deionized water or pure water).
5. For more detailed information, please refer to "LaserMach™ Pro user manual".