



**EVO VELOCITY TRIO Recommended parameters(1000W)**

Skin tones	Wavelength	Body area	Dark Hair				Brown Hair				Blond hair				
			Pulse width	Fluence	Frequency	Treatment times	Pulse width	Fluence	Frequency	Treatment times	Pulse width	Fluence	Frequency	Treatment times	
I-II	755nm	Face	Women Lip hair	75	25	1	6-8	85	30	1	6-8	100	35	1	8-10
			Women Hairline	70	20	1	6-8	80	25	1	6-8	90	30	1	8-10
			Men whiskers	70	20	1	4-6	80	25	1	4-6	90	30	1	4-6
		Arms	30	10	3-4	4-6	35	13	3-4	4-6	45	15	3-4	4-6	
		Shoulder/Back	30	10	3-5	4-6	35	13	3-5	4-6	45	14-15	3-5	4-6	
		Legs	30	10	3-4	4-6	35	13	3-4	4-6	45	15	3-4	4-6	
		Bikini	60	20	1-2	6-8	75	25	1-2	6-8	85	30	1-2	8-10	
		Underarms	30	10	3-4	4-6	35	13	3-4	4-6	45	15	3-4	4-6	
III-IV	808nm	Face	Women Lip hair	75	12	1	6-8	75	14	1	6-8				
			Women Hairline	70	10	1	6-8	70	12	1	6-8				
			Men whiskers	70	10	1	4-6	70	12	1	4-6				
		Arms	30	9	3-4	4-6	40	10	3-4	4-6					
		Shoulder/Back	30	9	3-5	4-6	40	10	3-5	4-6					
		Legs	30	9	3-4	4-6	40	10	3-4	4-6					
		Bikini	60	10	1-2	6-8	60	12	1-2	6-8					
		Underarms	30	9	3-4	4-6	40	10	3-4	4-6					
V-VI	1064nm	Face	Women Lip hair	100	12	1	6-8	140	13	1	6-8				
			Women Hairline	100	14	1	6-8	140	15	1	6-8				
			Men whiskers	100	14	1	4-6	140	15	1	4-6				
		Arms	100	6	3-4	4-6	100	8	3-4	4-6					
		Shoulder/Back	80-100	6	3-5	4-6	80-100	8	3-5	4-6					
		Legs	100	6	3-4	4-6	100	8	3-4	4-6					
		Bikini	100	14	1	6-8	100	15	1	6-8					
		Underarms	100	6	3-4	4-6	100	8	3-4	4-6					

Note: 1 the parameters are recommend ones, operator should adjust them depending on different patients' conditions.

2 If frequency is higher, which may occur high energy on same area, please adjust the moving speed by in-motion mode

3 Under the same Pulse, bigger pulse width could be less pain

4 Total output energy per unit time is: Fluence\*Frequency;

5 The paraters are suitable for ①The patients who is the first time to do laser hair removal treatment, NRS pain level below 4 ②New operator 6 During operation, please prefer to use the max. pulse which patients could bear.

7 After treatment, if there is some skin redness or scorching feeling, please do ice compress immediately.

8 The treatment times which up tp treatment end are bound up with pulse.