



EVO IPL SHR E-light Machine

SHR IPL Skin Procedures

SHR (SHR / SSR)

| Procedure | Handle | Energy (J/cm2) | Frequency (Hz) | Pulsh Width (ms) | Duration | Number of sessions | Interval between 2 treatments |
|-----------------------------------|--------|----------------|----------------|------------------|----------|--------------------|-------------------------------|
| hair removal dark skin&thick hair | SHR | 6 – 10 | 2 – 3 | 5 – 7 | 3 – 4 | 4 - 6 times | 28 days |
| hair removal light skin&fine hair | SHR | 8 – 13 | 3 – 4 | 5 – 7 | 3 – 4 | 4 - 6 times | 28 days |
| skin rejuvenation | SSR | 8 – 11 | 2 – 3 | 5 | 2 – 3 | 3 - 5 times | 21 days |

IPL (HR / SR)

| Procedure | Handle | Energy (J/cm2) | Speed | Pulse | Duration | Number of sessions | Interval between 2 treatments |
|--|--------|------------------------------------|-------|-------|----------|--------------------|-------------------------------|
| hair removal dark skin&thick hair | HR | 13 – 15 | 1 – 2 | 3 | 3 – 4 | 4 - 6 times | 28 days |
| hair removal light skin&fine hair | HR | 15 – 23 | 1 – 2 | 3 | 3 – 4 | 4 - 6 times | 28 days |
| skin rejuvenation | SR | dark skin 8-13 white skin 13-18 | 1 – 2 | 3 | 2– 3 | 3 - 5 times | 21 days |
| pigment removal freckle acne vascular | SR | dark skin 9-15 white skin 14-19 | 1 – 2 | 3 | 2 – 3 | 3 - 5 times | 21 days |

E-light (EHR / ESR)

| Procedure | Handle | IPL (J/cm2) | Speed | Pulse | Duration | Number of sessions | Interval between 2 treatments |
|--|--------|------------------------------------|-------|-------|----------|--------------------|-------------------------------|
| hair removal dark skin&thick hair | EHR | 13 – 15 | 1 – 2 | 3 | 3 – 4 | 4 - 6 times | 28 days |
| hair removal light skin&fine hair | EHR | 15 – 23 | 1 – 2 | 3 | 3 – 4 | 4 - 6 times | 28 days |
| skin rejuvenation | ESR | dark skin 8-13 white skin 13-18 | 1 – 2 | 3 | 3 – 4 | 3 - 5 times | 21 days |
| pigment removal freckle acne vascular | ESR | dark skin 9-15 white skin 14-19 | 1 – 2 | 3 | 2 – 3 | 3 - 5 times | 21 days |

**RF energy, we advise try from 10 for body, try from 8 for face.
Increase little by little according to patients feeling.
The best situation is patients feel warm, but not too hot.**

Six different filters

- 430nm: Acne
- 530nm: Skin resurfacing ;pigmented lesions
- 560nm: Wrinkle; vascular lesions
- 640nm: Light skin color fine hair removal
- 690nm: Brown skin color hair removal
- 755nm: Dark skin color hair removal

Treatment Notice

Before treatment

- Fully communicate with customers, understand their needs, physical conditions (whether) there are injections, whether skin care products containing hormones are used, whether they have been exposed to the sun recently), take photos, and fill in customer information files.
- Check whether the instrument is working properly and clean the lens before treatment.

Explain the feelings during treatment.

In treatment

- Apply the gel to the operation area, the gel needs to be spread evenly.
- Both operators and customers need to wear glasses.
- The light outlet of the treatment handle is gently attached to the skin, and always keeps it vertical to the skin (tilting can easily cause energy to concentrate on the epidermis, which is easy to burn.)
- Choose sliding or fixed-point treatment according to the area and treatment items (sliding treatment is recommended for large-area hair removal and small-area hair removal, skin rejuvenation, freckle and acne removal, red blood is recommended for fixed-point treatment.)
- During the operation, always pay attention to the skin reaction of the guests, and adjust the treatment parameters in time.

Quire about the client's feelings during the treatment process and communicate in time.

After treatment

- Apply repair products to repair in time.
- Explain the precautions after treatment and make an appointment for the next treatment time.
- Turn off the instrument and record the parameters used by the guests.