

Upgrade Process Instructions for M22 machine to Version 3.0.0.0 from Version 2.x.x.x

Preparing

1. Copy SW-10007530_A Update Pack to USB Flash Disk from your PC.

SW-10007530_A

[X:\ShersFolder\lumenis\m22 lightsheer\M22\usb convert from rev 2 to 3](#)

Before Starting

1. Turn on the M22 machine with a green push button on the front panel.
2. Enter user code: 1 1 1 1 + "Ok".
3. Picture software version in "Utility Menu" -> "Service" -> "Software Version" window.
4. Return to Hand Piece Selection Screen
5. Press ikon in right upper corner on the screen -> "Shut down".

Update Process

1. Insert USB Flash with up Pack to USB Port on back machine panel in upper right corner.
2. Double Click on the "My Device" icon in the left upper corner.
3. In "My Device" window - Double Click on USB Disk.
4. One click on SW-10007530_A Pack -> "Edit" -> "Copy".
5. Return to "My Device" window by pressing the back icon.
6. Doble click on "My Documents" -> "Edit" -> "Paste".
7. Wait for the coping process to finish.
8. Double Click on upgrade Pack
9. Double Click on "Lume2install" file.
10. Wait for the update process to finish.
6. Turn off the M22 machine with a green push button on the front panel.

Upgrade status check

1. Turn on the M22 machine with a green push button on the front panel.
2. Enter user code: 1 1 1 1 + "Ok".
3. Check the software version in "Utility Menu" -> "Service" -> "Software Version" window.