

Upgrade Process Instructions for M22 machine to Version 5.0.9.13 from Version 3.x.x.x and 4.x.x.x

Preparing

1. Copy SW-20096081 Pack to USB Flash Disk from your PC.

Before Starting

1. Turn on the M22 machine with a green push button on the front panel.
2. Enter user code: 1 1 1 1 + "Ok".
3. Picture software version in "Utility Menu" -> "Service" -> "Software Version" window.
4. Return to Hand Piece Selection Screen
5. Press ikon in the upper right corner on the screen -> "Shut down".

Update Process

1. Insert USB Flash with SW-20096081 Pack to USB Port on back machine panel in upper right corner.
2. Double Click on "My Device" icon in the left upper corner.
3. In the "My Device" window - Double Click on USB Disk.
4. One click on SW-20096081 Pack -> "Edit" -> "Copy".
5. Return to the "My Device" window by pressing the back icon.
6. Doble click on "My Documents" -> "Edit" -> "Paste".
7. Wait for the coping process to finish.
8. Double Click on SW-20096081 Pack
9. Double Click on "M22InstalationFiles"
10. Double Click on "Lume2"
11. Double Click on "Lume2install" file.
12. Wait for the update process to finish.
13. Turn off the M22 machine with a green push button on the front panel.

Upgrade status check

1. Turn on the M22 machine with a green push button on the front panel.
2. Enter user code: 1 1 1 1 + "Ok".
3. Check the software version in "Utility Menu" -> "Service" -> "Software Version" window.